

# Soluscope™ PAA



## Disinfectant for automated endoscope reprocessing

Disinfectant for flexible endoscopes. To be used exclusively with Soluscope Serie 4 PA, Soluscope Serie 5 and Soluscope Serie XS washer-disinfectors

- Peracetic acid-based disinfectant
- Broad spectrum antimicrobial activity
- Compliant with EN ISO 15883-4
- Compatible with a large range of flexible endoscopes
- Lock key system and cannula integrated to the container



# Soluscope™ PAA

Disinfectant for flexible endoscopes. To be used exclusively with Soluscope Serie 4 PA, Soluscope Serie 5 and Soluscope Serie XS washer-disinfectors



Soluscope Serie 4 PA



Soluscope Serie XS



Soluscope Serie 5

## Product composition

- 5% (w/w) Peracetic Acid
- Stabilising agents

## Product characteristics

- Shelf life: 24 months closed
- Aspect: clear and colourless
- Odour: characteristic
- pH pure product: 1 - 4.2\*
- Specific gravity at 20°C: 1.10 - 1.13
- Refractive index: 1.35 - 1.37
- Viscosity: < 10 mPa/s

\*pH value measurements under operating conditions by means of a pH meter with temperature compensation.

## Microbiological properties

| Claim/Conditions                                                                                                                                        | Concentration | Contact time | Temperature |
|---------------------------------------------------------------------------------------------------------------------------------------------------------|---------------|--------------|-------------|
| Soluscope Serie 4 PA (in WD) and Soluscope Serie 5 (in WD)                                                                                              |               |              |             |
| Bactericidal* (EN 13727, EN 14561),<br>Mycobactericidal* (EN 14348, EN 14563),<br>Fungicidal* (EN 13624, EN 14562) &<br>Virucidal* (EN 14476, EN 17111) | 1.8%          | 3 mins       | 40°C        |
| Soluscope Serie XS (in WD)                                                                                                                              |               |              |             |
| Bactericidal* (EN 13727, EN 14561),<br>Mycobactericidal* (EN 14348, EN 14563),<br>Fungicidal* (EN 13624, EN 14562) &<br>Virucidal* (EN 14476, EN 17111) | 1.0%          | 6 mins       | 40°C        |

Refer to the washer disinfectors' manual for conditions for use

\*=clean conditions

## Packaging and ordering information

| Product       | Packaging unit | Order code |
|---------------|----------------|------------|
| Soluscope PAA | 2 x 5 L        | 2206736    |

## Instructions for use

Concentrated solution

- Install the container
- Screw the grey cap connected to your Soluscope Serie 4 PA, Soluscope Serie 5 or Soluscope Serie XS onto the container
- Ensure that the vent hole is not blocked
- Automatic adjustment of cycle parameters by the Washer-Disinfector
- Refer to the Washer-Disinfector users' manual

Shelf life after opening: 2 months while connected to the washer-disinfector (write the opening date on the label in the location provided for this purpose).

## Storage

Warning – respect the precautions for use (drawn up according to the European rules in force regarding the classification and labelling of chemical products). Store between +5°C and +25°C.

## Medical device use safely

Class IIb

Please read label and product information.



FOR PROFESSIONAL USE ONLY.



**SOLUSCOPE SAS**  
(an Ecolab Company)  
100, Rue du Fauge  
Z.I Les Paluds - 13400  
Aubagne - France

Distributed by:

**ECOLAB EUROPE GmbH**  
Hofwiesenstrasse 349  
CH-8050 Zürich,  
Schweiz

**ECOLAB®**  
PROTECTING WHAT'S VITAL™

**Soluscope**